## **Caring for Your Scleral Contact Lenses**



Patient Name:			Date:	
, ,	ar your contacts for _ um wear time. <b>It is N</b>	,		rs per day until you have your contacts.
Recommended Solu	utions:			
Insertion: Preservat	tive Free Saline (Fill C	OVER the top of the I	ens)	
□ VibrantVue	□ LacriPure	□ ScleralFil	□ Nutrifill	□ Other
Daily Cleaner/Multi-	-Purpose Solution			
□ Tangible Clean	□ Unique PH	□ Boston Simplus	□ ClearCare	□ Other
Protein Removers*				
□ Boston One Step	□ Boston Advance (	Cleaner □ Progent	*Do NOT use if yo	our lenses are treated with Hydra-PEG
Rewetting Drops (us	se as needed)			
□ Refresh	□ Oasis Tears	□ Blink (	□ Systane	□ Other
Special Instructions	s:			
Follow Up Appointn	nent:	Please wear Contact	Lenses in for this app	pointment unless otherwise instructed
I understand the ins	tructions given to me	e for healthy and suc	cessful wear or	scleral contact lenses.
Patient Signature			Date	

## Do's & Don't's for Successful Wear of Contact Lenses

- Always wash and rinse your hands thoroughly before handling your lenses or touching your eye. Use a mild, fragrance free soap and dry your hands with a lint-free cloth or tissues.
- Oclean your lenses after removal as directed. Use only the recommended solutions.
- Clean your lens storage case in warm soapy water at least once a week. Rinse thoroughly and allow to air-dry on a clean paper towel.
- When handling your lenses, work over a clean flat surface. Plug the drain if working over a sink.
- Insert your lenses before applying make-up to avoid getting residue on or behind the lenses.
- ✓ If you drop a lens, clean and rinse with multi-purposes solution before reapplying.
- Carry your storage case and solution with you at all times and keep your glasses up to date if for any reason you can't wear your lenses.
- Close your eyes when using cosmetics or aerosol sprays, such as hairsprays, while wearing your lenses. Such products may leave a residual film on the lens.
- Wear sunglasses for outdoor activities and safety glasses in any dusty environment.
- 🗴 Do not continue wearing your lenses if your eyes are uncomfortable or unusually red.
- Do not place a lens in your mouth for cleaning or wetting.
- 🗴 Do not use tap water to soak or rinse your lenses.
- 🗴 Do not rub your eyes vigorously while wearing lenses.
- (x) Do not mix up your lenses. The best way to avoid this is to follow an insertion, removal and storage routine every time you wear and remove your lenses.
- 🗴 Do not expose your lenses or storage case to undue heat.