

HOA (WAVEFRONT) CORRECTED ORDERING PROCESS

Visionary Optics
uniquely specialized contact lenses

powered by
OVITZ® xwave II

1

FINALIZE PATIENT LENS FIT

Find your patient's optimal lens fit with the help of the Visionary Optics Team.

2

ORDER MARKED LENS ("BASE LENS WITH MARKINGS")

Once your fit is finalized, order your **Marked Lens** including a spherical equivalent over-refraction.



Contact our HOA Consultation Department
at HOAconsult@visionary-optics.com.

3

OBTAIN HOA SCAN OVER MARKED LENS

Before proceeding with the OVITZ® xwave measurements, verify that the marked lens is stable and providing an optimized fit. An additional **Marked Lens** can be ordered with fit adjustments only if clinically necessary.

Settling Time

It is important to allow the marked lens to settle as much as possible before taking the OVITZ® xwave measurements. We recommend a minimum settling time of at least 3 hours. Since the **Marked Lens** is a duplicate of the patient's optimally fitting lens, we suggest having the **Marked Lens** shipped directly to your patient and asking them to wear it for at least 3 hours prior to coming in for their OVITZ® xwave measurements.

Take the OVITZ® xwave measurement over the **Marked Lens** following the guidelines below:



- Apply lens and let settle for as long as possible (**minimum of 3 hours preferred**). Ensure that the black insertion dot is at the bottom half of the lens. Dots may rotate slightly.

Reminder: All Visionary Optics HOA lenses will have 1 black dot on the right eye and 2 black dots on the left eye.

- Select the "Base Lens with Markings" measurement option.
- Select Visionary Optics and choose your lens type (Latitude or Europa).
- Make sure to keep the room dark in order to maximize the pupil size.
- Have the patient cover their fellow eye using the occluder and look straight into the red dot.
- Follow the OVITZ® xwave software prompts to finalize the lens order.
- Click the "Submit Lens Order" button on the measurement results page.



HOA files are sent to OVITZ®
for review & verification.



OVITZ® will then send the files to
Visionary Optics to manufacture.

4

RECEIVE HOA CORRECTED ("FINAL HOA LENS") LENS | VERIFY VA | DISPENSE

Verify optimal fit & visual performance. Use the OVITZ® xwave system to take the "Final HOA Lens" measurement and indicate if an additional over-refraction or other adjustments are required to achieve optimal vision. Requests and questions can be noted directly in the order submission.

For troubleshooting vision, contact the OVITZ® Team at support@ovitz.us or 833.684.8987.

All Visionary Optics HOA lenses will be manufactured with black insertion dots designed to be applied to the patient's eye at or around 6 o'clock. There will be 1 dot on the right eye (OD) and 2 dots on the left eye (OS). The dot(s) should remain stable after settling. It is ok if the dot(s) is not exactly at 6 o'clock as long as the lens is stable and not moving around.



HOA-Corrected Technology Available on:

